Hyris System™ 16-wells Cartridge Instruction for Use (IFU) ENGLISH, ITALIANO, FRANÇAIS

P/N: HyCT16.01

Version 2.1

Milan, September 2021

Hyris Srl

Corso Garibaldi, 60
20121 Milano, Italy
Phone: +39.02.82951302
Mail: office@hyris.net

Other contacts:

Hyris Ltd

Lower Ground Floor, One George
Yard, ECSV 9DF, London UK
Phone: +44.2039361227
Mail: office.uk@hyris.net

Hyris Asia Pac

38 Ang Mo Kio Industrial Park 2 #02-07A
569511 Singapore
Phone: +65.8160.7207
Mail: office.uk@hyris.net

US West Coast Sales Inquiries
Los Gatos

Mail: office.us@hyris.net

INDEX

ENGLISH

HYRIS 16-wells cartridge	2
Manufacturer	2
Instructions for use	2
Warning and precautions	2
Symbols	2

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ENGLISH
HYRIS 16-wells cartridge
Manufacturer

Hyris Srl

Corso Garibaldi, 60 20121
 Milano (MI) Italy

Visit our website at
www.hyris.net

Ordering information

Email: office@hyris.net

Phone: +39.02.82951302

Technical assistance

Email: support@hyris.net

Phone: +39.02.82951302

Product information:

Email: info@hyris.net

Intended use

The 16-wells cartridge HyCT16.01 is intended to be used as an accessory to the HYRIS bCUBE™ instrument for the in vitro qualitative detection of nucleic acids.

Instructions for use

Open the external wrapping of the cartridge using the appropriate PPE and precautions to avoid contamination. Load the cartridge according to the HYRIS bKIT™ Instructions for use. Seal the cartridge with the aluminium adhesive black seals. Load the cartridge in the HYRIS bCUBE™.

Warning and precautions

- For in vitro diagnostic use
- Store the product in the original packaging, at room temperature
- Use only the aluminum adhesive black seals provided with the cartridges to seal the cartridge, taking care to ensure the complete adhesion of the adhesive side to the metallic surface of the cartridge
- Avoid touching the adhesive side of the aluminum foil that will be in contact with the HYRIS cartridge
- Initiate the run within ten (10) minutes from cartridge loading
- Do not attempt to remove the adhesive foil cover from the cartridge after use or attempt to re-use a cartridge or reagents that have been used in previous runs
- Dispose of the used cartridges according to local, state, and federal regulations
- Treat all human specimens and all consumables, including HYRIS cartridge, that have come into contact with the samples as capable of transmitting infectious agents

Symbols

SYMBOL	DESCRIPTION
	Part number
	Batch number
	Date of manufacture
	Manufacturer
	Non-sterile
	EC Directive 98/79 compliant

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